

Saliva Collection Kits have been designed for collecting superior samples which used for genetics analysis, and now, they have been used for Covid-19 testing widely. We can collect the sample by ourselves at home, this is the safe way for collecting virus sample. MandeLab's VTM is perfect for collection, stabilization and transportation of the Covid-19 and Influenza H1N1 samples.




Collection Instructions

Please don't eat, drink, smoke or chew gum 1 hour before giving your saliva samples.

1. Rinse mouth by clear water to clean intraoral residuum 30 minutes before the collection;
2. Raise the tongue tip against the palate or jaw tooth root for saliva enrichment, spit into the funnel until the amount of liquid saliva (not the bubbles) reaches the scale line.
3. For MSK-SCK003, hold the tube upright and then close the funnel lid by pushing the lid. The liquid will be released into the tube to mix with saliva, make sure that the lid is closed tightly. Finally, unscrew the funnel from the tube, use the other cap to close the tube tightly.
4. For MSK-SCK004, please keep the tube upright with one hand and then screw off the funnel with the other hand. Finally, use the other cap to close the tube tightly.

5. Shake the capped tube for around 10 seconds, and then discard or recycle the used funnel.

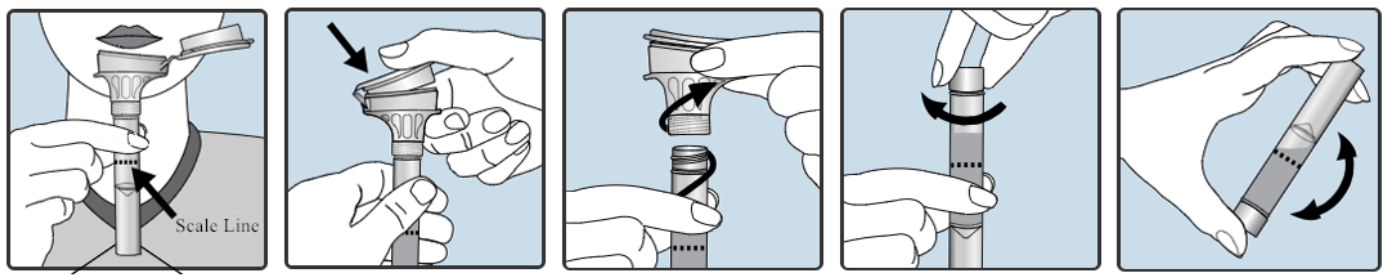

Product Features

- A new way for the Covid-19 collection;
- High quality, and high quantity samples;
- Non-invasive, easy collection and convenient transport;
- RNA is stable for over 2 years at the ambient temperature;
- Optimized for manual sample processing and automated workflows.

Product Pictures



Price Information

Item	Kit No.	Kit Descriptions	Standard	Customized	Packing Information
1	MSK-SCK003	SCK with removable Funnel VTM	\$1.50	Negotiable	200 Pcs per carton, the carton dimension is 380*350*475 mm.
2	MSK-SCK004	SCK with All-in-one System VTM	\$1.60	Negotiable	480 Pcs per carton, the carton dimension is 530 ×650 ×460 mm.