

MSK-HPV004 Sampling Kits have been mainly used for collection, stabilization and transportation of the cervical exfoliated cells before HPV screening. Normally, a sampling kit contains 1 cervical sampling swab, 1 collection tube with HPV transport medium, and 1 collection bag. We also provide HPV self-sampling solutions, customization is welcome.



HPV Sampling Kit Contains


Disposable Sampling Swab(s)

MS-VF4503 is a Flocked Sampling Swab used for cell & virus specimen collection, it has been designed for HPV&TCT specimen collection, and Gynecological clinical diagnostics. It utilizes state of the art “spray on technology” that the flocking process by means of an electro-static charge perpendicularly attaches millions of nylon microfibers on the medical grade handle tip. The flocked swab is ideal for collecting large amount of cells and rapid elution of the specimens that instantly releases the cells into the transport medium. It has been well recognized and adopted by the diagnostic test kit manufacturers who produce reagents in molecular genetics, forensics and clinical laboratories sectors. The perpendicular nylon fibers act like a soft brush, allowing the improved collection and release of both cellular and liquid samples.

HPV Transport Medium

The HPV transport medium is mainly composed of HBSS, Preservatives, Antibiotics, Ethylene Diamine Tetraacetic Acid, Tritonx-100 and Methanol. It is used to preserve and transport the collected cells, and only for in vitro analysis and testing, not for therapeutic use.

How to Use the HPV Sampling Kit?

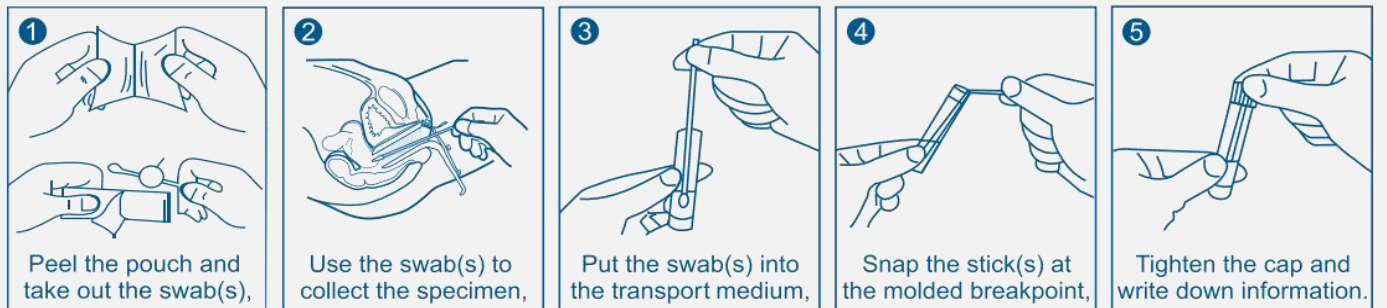

Storage and Disposal

The optimum storage temperature is 2°C to 30 °C. Do not incubate, or freeze prior to use, improper storage will result in a loss of testing. Medical waste must be disposed of in compliance with the local legislation. Please take the appropriate precautions for infected material if necessary.

Product Pictures

